

Q: What is Detachol[®]?

A: Detachol is a non-irritating adhesive remover used to aid in the removal of dressings, tapes and most sticky residue from the skin and other surfaces.

Q: What the ingredients of Detachol?

A: Specially Refined Paraffin Hydrocarbons

Q: What is Detachol used for?

A: Below please find a list of hospital departments in which Detachol is frequently used:



Pediatrics	Wound Dressing Removal, IV Dressing Removal, Catheter Securement Device Removal, ET & NG Tube Removal
Hematology/Oncology	IV Dressing Removal, Wound Dressing Removal
Intensive Care Units	IV Dressing Removal, Catheter Securement Device Removal, Wound Dressing Removal, ET & NG Tube Removal
Emergency Room	Dressing Changes
Operating Room	Tape and Adhesive Residue Removal, ET Tube Removal
Wound Ostomy	Wound Dressing Removal, Removal of Ostomy Appliances, Removal of NPWT Drapes
Respiratory Therapy	ET Tube & NG Tube Removal
IV Therapy	IV Dressing Removal, Catheter Securement Device Removal
Sleep Lab/Telemetry	Lead and Residue Removal

Q: What are the unique benefits of Detachol?

A: Unique benefits include:

Gentle Removal	Alcohol- and acetone-free product that improves patient comfort.
Reduced Risk	Easy adhesive removal helps reduce the chance of skin tears or skin irritation.
Convenience	A convenient easy-to-use vial will do the job of multiple remover wipes, saving health care professional time.
Complete Removal	With Detachol, adhesive residue in which bacteria can breed, will not be left behind.
Mild Scent	Will not irritate patients that may be very sensitive to harsh odors.

Q: How does Detachol compare to other adhesive removers?

A: Detachol will not dry the skin. It will not degrade nurses' gloves like other alcohol-based adhesive removers. It comes in a variety of convenient sizes, including single-use vial packaging. The vial effectively distributes the remover to the desired areas. Nurses report that numerous remover wipes are needed to accomplish the removal of a simple dressing. Whereas, a Detachol vial will easily melt the adhesive away. This saves time and reduces patients' pain and anxiety.

Q: How is Detachol supplied?

A: Detachol is available in several configurations:

Description	Unit	HRI #	Internal Order#
Detachol Vials	48 x 2/3 mL	0496-0513-48	
Detachol Vials	24 x 1.6 mL	0496-0513-24	
Detachol Unit Dose	15mL bottle with plug	0496-0513-15	
Detachol Dispenser Cap	2oz bottle	0496-0513-06	
Detachol Dispenser Cap	4oz bottle	0496-0513-04	

